

# ROM Elbow Braces

## Post Operative Bracing Solutions

### Routinely Prescribed For...

Fixed or controlled ROM in the treatment of post elbow surgery: elbow reconstruction, ligament and tendon repairs, surgical repair of a fracture and biceps repair.

### These Products Can Be Ordered...

- Off-The-Shelf in a variety of models & lengths

### Standard Features...

- Tool Less Adjustment (ROM-LBO Model)
- Drop Lock With Free Motion (ROM-LBO Model)
- One Size Fits All (ROM-LBO Model)
- ROM Hinge(s) With Extension & Flexion Stops
- Adjustable Aluminum Bars & Straps

### Available Options Include...

- Breeze Pads Or Individual Wrap Pads (ROM-LBO Model)
- Single Or Dual Hinge Models
- Shells & Hand Extension (dual upright, Low Profile model)



The new ROM-LBO elbow brace features tool-free length adjustment, bar extension plates that can be contoured to the arm, multiple flexion and extension stop settings, a drop lock that can fixate the arm at 90 degrees, and Breeze pads.



The single or dual hinge Multi-Adjustable ROM Elbow features multiple flexion and extension stops, and Air pads.



The dual hinge Low Profile Elbow features adjustable shells, a low profile hinge with flexion and extension stops, and an optional hand extension.

## General Information

Ordered by : \_\_\_\_\_

Phone: \_\_\_\_\_

Patient's Name: \_\_\_\_\_

BILLING: P.O. Number \_\_\_\_\_

Townsend Account Number: \_\_\_\_\_

Bill To: \_\_\_\_\_

Address: \_\_\_\_\_

City: \_\_\_\_\_

State: \_\_\_\_\_ Zip Code: \_\_\_\_\_ Country: \_\_\_\_\_

Phone: \_\_\_\_\_ Fax: \_\_\_\_\_

Ship To: \_\_\_\_\_

Address: \_\_\_\_\_

City: \_\_\_\_\_

State: \_\_\_\_\_ Zip Code: \_\_\_\_\_ Country: \_\_\_\_\_

Phone: \_\_\_\_\_ Attention: \_\_\_\_\_

**Fit Date: If known, please indicate the date you are scheduled to fit the patient:** \_\_\_\_\_

### Shipping Preference:

Ground  2-Day P.M.  2-Day A.M.

Next Day P.M.  Next Day A.M.

(If no preference is indicated, this order will be shipped 2 Day P.M.)

**Note:** We do not ship new or repaired products directly to patients.

## Sizing Reference

Measure from the wrist up to the elbow crease. If this measurement is eight inches or less, then a Small/Medium size or 15 inch model should be ordered. If the measurement is longer than eight inches, then it is best to order the Large/X-Large size or 16 inch model.

## Ordering Instructions

Please complete and fax this form to 800.798.2722 (24-hours a day). If you are calling in your order, this form indicates the options and information that will be required by our staff. For phone orders, please call 800.700.2722 between 7:00 a.m. and 4:00 p.m. (PST).

## ROM Post-Op Elbow Braces

### ROM-LBO

The LBO comes standard with four straps, tool less adjustability, 8 individual Breeze pads or 4 individual full wrap pads; single or dual upright applications. Easy adjustable flexion and extension stops provide range of motion control in 10 degree increments. The LBO can be locked at 90 degrees for post-surgical protocols. For single hinge applications, right or left arm must be indicated.

#### Single Hinge (SH) Model

Quantity: \_\_\_ LEFT ARM

Quantity: \_\_\_ RIGHT ARM

#### Pad Options:

Breeze Pads

Individual Full Wrap

#### Dual Hinge (DH) Model

Quantity: \_\_\_ UNIVERSAL

#### Pad Options:

Breeze Pads

Individual Full Wrap



### Multi-Adjustment ROM Elbow

This brace has four straps and individual black pads that fit under each strap. Single hinge/bar and dual hinge/bar models can be ordered with a 15 or 16 inch bar length. Flexion and extension stops are in 10 degree increments and can be set without tools. Locking is possible by offsetting the flexion and extension stops at the same degree. For single hinge braces, you must indicate right or left arm.

#### Single Hinge (SH) Model

Quantity: \_\_\_ LEFT ARM Length: \_\_\_ 15 inch \_\_\_ 16 inch

Quantity: \_\_\_ RIGHT ARM Length: \_\_\_ 15 inch \_\_\_ 16 inch

#### Dual Hinge (DH) Model

Length (Indicate quantity): \_\_\_ 15 inch \_\_\_ 16 inch

Townsend's Multi-Adjustment ROM Elbow Brace features adjustable flexion and extension stops.



### BRACES WITH LOW PROFILE HINGES

Flexion can be limited at 90, 110 and 120 degrees, and extension can be stopped at 0, 15, 30, 45 and 90 degrees. The screw stops are adjusted using a small flathead screwdriver. The hinge can be locked at 90 degrees.


### Low Profile ROM Elbow (Dual Hinges) With Shells And Optional Hand Extension

The DUAL HINGE (DH) model is fabricated with rigid ABS plastic shells. This brace can be ordered with an optional bar and hand extension for increased stabilization and rotation control. The shells conform to the arm when the straps are tightened. Also, the length of the hand extension and position of the shells is adjustable.

Quantity: \_\_\_ S/M \_\_\_ L/XL

Install Optional Hand Extension? \_\_\_ Yes \_\_\_ No

Include Shoulder Sling Strap?\* \_\_\_ Yes \_\_\_ No

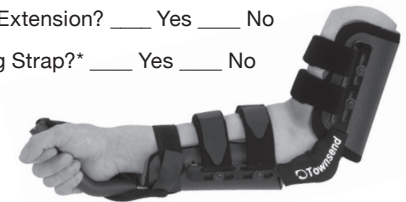

## Townsend Design

4615 Shepard St., Bakersfield, CA, 93313  
Phone: 800.432.3466 or 661.837.1795; Fax: 800.798.2722

Special Instructions: \_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_